


CONTROL PANEL

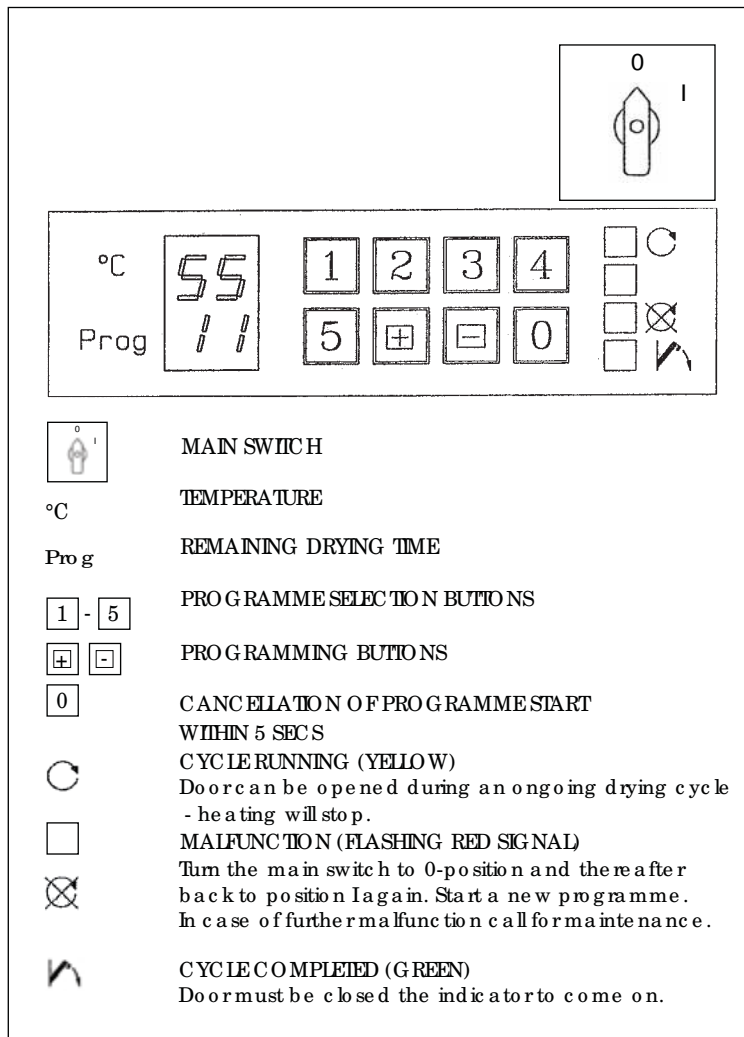

HOW TO USE DRYING CABINET

- TURN MAIN SWITCH TO POSITION I**
- OPEN DOOR AND PLACE GOODS IN.**
 AN-hose cassettes into the rails at the top in front.
CLOSE DOOR.
- SELECT PROGRAM 1-5**
 Start program by pushing the program button. Yellow indicator will come on. PROG indicator shows the remaining drying time. The door can be opened during a cycle, but heating will be interrupted. By pushing the button + during the cycle PROG indicator shows the program going on P1 ... P5.
- GREEN SIGNAL- PROGRAM COMPLETED, THE DOOR CAN BE OPENED.**
 Indicator lights up only when the door is closed.

⚠ NOTE! CHAMBER WALLS ARE HOT!

- WHEN THE CABINET IS NOT BEING USED PLEASE SWITCH OFF BY THE MAIN SWITCH.**

In pass-through models (with double doors) yellow signal lights at "clean side" during the cycle. When the cycle ends yellow signal at "clean side" switches to green and at "unclean side" both yellow and green are lit at the same time. After opening the door at "clean side" the yellow indicator switches off and at "unclean side" only the green light is on.